



PHILIPS

Automated
External Defibrillator

HeartStart FRx

Side by side.
Step by step.

Philips HeartStart FRx AED

Expert guidance. Intuitive support.

Philips HeartStart FRx AED

- Includes features to help guide the treatment of suspected sudden cardiac arrest (SCA) with easy setup, real-time metronome and voice commands paced to your actions
- Provides CPR instructions for infants and children under 25 kg (55 lb) or 0–8 years old, and adults and children over 25 kg (55 lb) or greater than 8 years old
- Has an optional Infant/Child Key; simply insert it and the defibrillator adjusts instruction and therapy eliminating the need for additional infant/child pads
- Includes pre-connected SMART Pads II that can be used for both adults and children; SMART Analysis automatically assesses heart rhythm and only advises a shock if the victim's rhythm is determined to be treatable by the Philips advanced algorithm
- Conducts a series of automatic self-tests daily, weekly and monthly, to check pad readiness and verify functionality and calibration of circuits and systems
- Features eight realistic SCA scenarios; simply insert the Training Pads II (available separately) to temporarily disable the FRx's energy delivery capability and the defibrillator will switch into training mode



Patented technology. Proven therapy.

Real-time, step-by-step voice commands paced to your actions, and an audible metronome and CPR guidance assist the responder. When treating an infant or child, simply insert the optional Infant/Child Key and the FRx adjusts instructions and therapy.



Designed to work where you need it

Lightweight, rugged and reliable, the FRx can withstand rough handling, extreme temperatures, and dusty or wet surfaces. Designed for use in harsh settings, it can withstand up to 500 kg (1,100 lb) and drops from 1.2 m (4 ft).



Ease of use

Pre-connected SMART Pads II can be used for both adults and children. Once installed and activated, the FRx is easy to maintain. It performs a series of automatic self-tests daily, weekly and monthly to check pad readiness and verify functionality and calibration of circuits and systems. It can last up to four years between battery replacements.

Philips HeartStart FRx AED

Patented Quick Shock feature allows FRx to typically deliver a shock within eight seconds after CPR.¹



1

Press the green On/Off button, which activates voice instruction and visual icons.



2

Place the pads on the patient as directed.



3

When advised by the device, press the orange Shock button.



1. Nichol G, Sayre MR, Guerra F, Poole J. Defibrillation for Ventricular Fibrillation: A Shocking Update. Journal American College of Cardiology. 2017;70(12),1496-1509. DOI: 10.1016/j. jacc.2017.07.778.



Philips Medical Systems Nederland B.V.
Veenpluis 6
5684PC Best



The Netherlands



0123



Philips Medical Systems
22100 Bothell Everett Highway
Bothell, WA 98021-8431, USA

© 2021 Koninklijke Philips N.V. All rights are reserved.
Philips reserves the right to make changes in specifications
and/or to discontinue any product at any time without notice
or obligation and will not be liable for any consequences
resulting from the use of this publication. Trademarks are the
property of Koninklijke Philips N.V. or their respective owners.

www.philips.com

Printed in the Netherlands.
4522 991 64051 * JUN 2021